


Intended use

The Boule Enzymatic Cleaner 100 mL is intended for use with the instrument systems given below for corrective and preventive cleaning of inlets and tubing in contact with blood (protein).

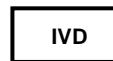

Summary and principles

The Boule Enzymatic Cleaner is a cleaner containing a proteolytic enzyme in a color coded (blue) buffer solution with a low content of salt and added surfactants. The enzyme and surfactants will dissolve protein and clots including those attached to surfaces facilitating the removal during rinsing. Using this cleaning solution at the stated regular intervals will ensure low background counts (PLT) and longer time between failures.

Storage and stability

The Boule Enzymatic Cleaner can be transported at ambient temperature (avoid freezing and allow adaption to room temperature before use). Prolonged storage above 30 °C may result in lessened cleaning activity of the product due to auto-degradation of the enzyme. Always store the bottles in an upright position to avoid any leakage from the cap. The cleaner is stable at least up to the expiration date as shown on the label when stored at a temperature of 2-30 °C.

Instructions for Use

A detailed description of use of the Boule Enzymatic Cleaner can be found in the instrument User's Manuals

Medonic CA 530/620 instruments

Medonic M-series instruments

Swelab AC and AC eo+ instruments

Swelab Alfa Instruments

Exigo instruments

Exigo EOS instruments



Precautions

- For In Vitro diagnostic use only on instruments as stated above.
- This product should not be disposed of in general waste. The cleaner should be disposed of according to local regulations. When draining remaining solution after full use through the sewage system, always use plenty of water. The bottle can be recycled after rinsing.
- Solution passing through the instrument during cleaning will be collected and disposed of as a part of the general waste from the instrument.
- This product is intended for use as supplied. Adulteration by dilution or addition of any materials to the product invalidates any diagnostic use of the product.

Ordering information and service

For international customers contact your local distributor for orders and support.

REF	Description	Packaging
1504112	Boule Enzymatic Cleaner	100 mL